Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Istruzioni per l'uso
Gebruiksaanwijzing

DAHO® Typ 2.6 DAHO® Typ 2.D

Version: 2, Stand: 30.09.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	11
FR - Mode d'emploi	18
IT - Istruzioni per l'uso	25
NL - Gebruiksaanwijzing	32

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller
Manufacturer
Fabricant
Produttore
Fabrikant

Herstellungsdatum
Date of manufacture
Date de fabrication
Data di produzione
Productiedatum

Artikelnummer
Article number
Numéro d'article
Numero articolo
Artikelnummer

Achtung
Attention
A noter
Attenzione
Achtung

einzelner Patient - mehrfach anwendbar
single patient - multiple use
patient individuel - utilisable plusieurs fois
singolo paziente - utilizzabile più volte
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge
Production lot number, batch
Numéro de lot de fabrication, lot
Numero di lotto di produzione, partita
Productienummer, batch

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.

1. Intended Use

Hand orthoses are used for immobilization, mobilization and/ or correction.

2. Technical data

2.1 DAHO® Typ 2.6

Extension orthosis with rigid wrist connector

Order No.	Size	Side	Article No.
DAH026-SL	S	left	4 695 020 01 01 000
DAH026-ML	M	left	4 695 020 01 02 000
DAH026-LL	L	left	4 695 020 01 03 000
DAH026-XLL	XL	left	4 695 020 01 04 000
DAH026-SR	S	right	4 695 020 02 01 000
DAH026-MR	M	right	4 695 020 02 02 000
DAH026-LR	L	right	4 695 020 02 03 000
DAH026-XLR	XL	right	4 695 020 02 04 000

Enclosed Spring-lengths:

Size	Thumb*	Index finger	Middle finger	Ring Finger	Little Finger
S	-	12 cm	14 cm	12 cm	12 cm
M	2 x 12 cm	14 cm	16 cm	14 cm	12 cm
L/XL	2 x 14 cm	16 cm	16 cm	16 cm	14 cm

*valid for the separate Thumb Extension 2.6

2.2 Individual and spare parts DAHO® Typ 2.6

2.2.1 Dynamic Thumb Extension

Order No.	Size	Side	Article No.
DAHO-EXML	S / M	left	4 695 029 01 06 033
DAHO-EXLL	L / XL	left	4 695 029 01 06 044
DAHO-EXMR	S / M	right	4 695 029 02 06 033
DAHO-EXLR	L / XL	right	4 695 029 02 06 044

2.2.2 Midhand Cuff

REF

Order No.	Size	Side	Artcile No.
E-DAH068	S	left	4 695 029 01 01 022
E-DAH069	M	left	4 695 029 01 01 033
E-DAH070	L	left	4 695 029 01 01 044
E-DAH085	XL	left	4 695 029 01 01 055
E-DAH071	S	right	4 695 029 02 01 022
E-DAH072	M	right	4 695 029 02 01 033
E-DAH073	L	right	4 695 029 02 01 044
E-DAH086	XL	right	4 695 029 02 01 055

2.2.3 Finger loop set

REF

Order No.	Size	Unit	Artcile No.
E-DAH074	universal	5 pieces	4 695 029 00 02 000

2.2.4 Loop padding set

REF

Order No.	Size	Unit	Artcile No.
E-DAH075	universal	5 pieces	4 695 029 00 03 000

2.2.5 Finger base with screws

REF

Order No.	Size	Unit	Artcile No.
E-DAH076	universal	1 pieces	4 695 029 00 04 000

2.2.6 Silicone tubing

REF

Order No.	Size	Unit	Article No.
E-DAH077	universal	6 pieces	4 695 029 00 05 000

2.2.7 Leafspring set

REF

Order No.	Size	Unit	Article No.
E-DAH087	10 cm	3 pieces	4 695 029 01 06 033
E-DAH078	12 cm	3 pieces	4 695 029 01 06 044
E-DAH079	14 cm	3 pieces	4 695 029 02 06 033
E-DAH080	16 cm	3 pieces	4 695 029 02 06 044

2.3 DAHO® Typ 2.D

Extension orthosis with leaf springs without forearm cuff

REF

Order No.	Size	Side	Article No.
DAH02D-ML	M	left	4 695 030 01 02 000
DAH02D-LL	L	left	4 695 030 01 03 000
DAH02D-XLL	XL	left	4 695 030 01 04 000
DAH02D-MR	M	right	4 695 030 02 02 000
DAH02D-LR	L	right	4 695 030 02 03 000
DAH02D-XLR	XL	right	4 695 030 02 04 000

The Leafspring unit for DAHO® 2D must be ordered separately.

REF

Order No.	Side	Article No.
E-DAH088	10 cm	4 695 038 00 01 022
E-DAH089	12 cm	4 695 038 00 01 033
E-DAH090	14 cm	4 695 038 00 01 044
E-DAH091	16 cm	4 695 038 00 01 055

Contains:

1 Finger base (E-DAH076)
1 Looppadding (E-DAH075)
Fingerloop 2.D (E-DAH074)
Silicone tubing (E-DAH077)
and the chosen Leafspring

2.4 Individual and spare parts DAHO® Typ 2.D

2.4.1 Padding Handcuff

REF

Order No.	Size	Side	Article No.
E-DAH093	M	left	4 695 039 01 02 033
E-DAH094	L	left	4 695 039 01 02 044
E-DAH095	XL	left	4 695 039 01 02 055
E-DAH097	M	right	4 695 039 02 02 033
E-DAH098	L	right	4 695 039 02 02 044
E-DAH099	XL	right	4 695 039 02 02 055

3. Indications/ Contraindications

Indications:

- Early movement treatment after extensor tendon surgery
- Grip training after neurological defects with mild spasticity (type 2.6 only)
- Contracture prophylaxis for peripheral nerve lesions
- Redression of soft tissue contractures with flexion deformity
- Contractures around the finger joints (e.g. Dupuytren's disease) (type 2.D only)
- Improvement and development of grip strength (type 2.D only)

Contraindications:

- Extremely deformities of the forearm or hand
- Extremely small or large hands
- Open wounds (without wound care)
- severe spasticity

4. Side effects

If worn for a long time, contact and the corrective effect of the orthosis can cause marks and irritation on the skin.

5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of the medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- The DAHO® Hand orthosis is a prescription product that should be worn according to the medical instructions.
- DAHO® Hand orthosis should only be worn in accordance with these instructions and for the indications described.
- In the event of incorrect use product liability is excluded.
- In the event of over sensitivity of the skin, known contact allergies and sensitivity to pressure, a cotton glove should be worn under the DAHO® Hand orthosis.
- No inexpert alterations may be made to the product. This may result in the performance of the product being limited, with the consequence that product liability is excluded.
- Patients should consult their doctor in the event of unusual changes in the hand function (e.g. an increase of the number of complaints).
- Combination with other products must be discussed with the patient 's doctor beforehand.
- In the event of changes in performance or malfunctions of the orthosis, the patient should contact their orthopedic technician or doctor immediately.
- Avoid contact of the orthosis with wounds and skin deformations, without wound coverage!

6. Residual risks


When using the hand orthosis (outside of clothing) it can get caught/ tangled with other objects. Despite proper use, skin irritation and pressure points can occur in individual cases.

7. Application information

7.1 Size Selection


To select the right orthotic size, the metacarpal width is measured at the level of the MCP joints.

Size	Midhand width
S	5,5 – 7,0 cm
M	7,1 – 8,5 cm
L	8,6 – 11,0 cm
XL	9,0 – 11 cm For powerful hands

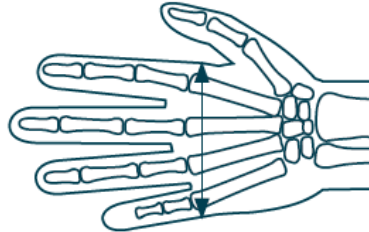Select the spring length:

Fingerlength	Spring
5 - 7 cm	10 cm
7 - 9 cm	12 cm
9 - 11 cm	14 cm
11 - 13 cm	16 cm

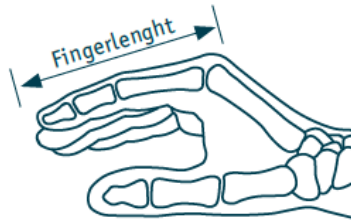7.2 Adjustment

The DAHO® Type 2.6 and Type 2.D are not ready-made products but a medical device that has to be individually adjusted to the patient's demands.

7.2.1 Test and possible exchange of the leaf springs according to the length of the fingers:


The selected spring units are attached at designated screw points. The barb of the spring pad ensures a stable directional adjustment of the spring. Each leaf spring has to be adjusted to the length of the fingers. In extended position the leaf spring should be approx. 1 cm longer than the finger (four different lengths are available which can be exchanged when needed).

The finger loops should be close to the DIP joints. To have enough sliding range the finger loops should be close to the proximal end of the groove in extended position. If a different position of the finger loop is desired the length of the spring must be adjusted. Loosen the attachment screw for adjustment. The spring action can be enhanced by a second leaf spring of the same length is inserted into the spring unit.


7.2.2 Adjustment of the finger loops:

The opening of the finger loop should be just that wide so that the finger can glide inside without opening the hook-and-loop fastener.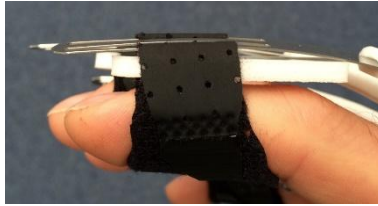

The size of the opening is adjusted once and then the remaining hook-and-loop fastener should be cut off.

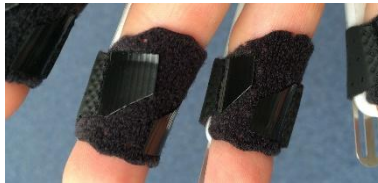
7.2.3 Loop cushion:

When fitting short fingers, the loop cushion should be shortened to avoid that the loop cushion push the finger out of the finger loop during full extension of the finger.

7.2.4 Adjustment of the forearm and the midhand cuff (only Type 2.6):

DAH0 2.6 will be supplied with a fixed connection bar in neutral position which can be adjusted if needed. Do not bend sharp edges (risk of fracture). For a flexed position of the hand the long connection bar E-DAH060 should be used. In addition, the wrist coil spring E-DAH043 can be used if free motion and a support of the wrist extension is desired. The plastic shells can be thermoplastically or mechanically deformed.

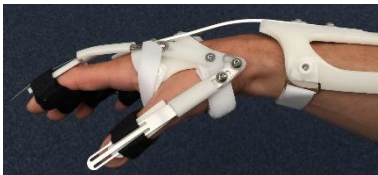8. Maintenance and Cleaning

The orthosis can be washed with mild soap and hand warm water.
Commercially available disinfectants should be used for surface disinfection.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

10. Warranty and Service life

We recommend to change/ renew the hand orthosis after 6 month.

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

The orthosis has no special storage regulations.
Please note the country-specific disposal.

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