Gebrauchsanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

ST80, ST81, ST82

Nylon-Kompressionshüllen
Nylon Compression Sheaths

Version: 4, Stand: 05.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

| | |
|--|----|
| Symbole/ Icons/ Symboles/ Simboli..... | 3 |
| DE - Gebrauchsanweisung..... | 4 |
| EN - Instruction Manual | 8 |
| FR - Mode d'emploi | 12 |
| IT - Istruzioni per l'uso | 16 |
| NL - Gebruiksaanwijzing | 20 |

Symbole/ Icons/ Symboles/ Simboli

| | | | |
|---|--|---|--|
|  | <p>Hersteller Manufacturer Fabricant Produttore Fabrikant</p> |  | <p>Herstellungsdatum Date of manufacture Date de fabrication Data di produzione Productiedatum</p> |
|  | <p>Artikelnummer Article number Numéro d'article Numero articolo Artikelnummer</p> |  | <p>Achtung Attention A noter Attenzione Achtung</p> |
|  | <p>Fertigungslosnummer, Charge Production lot number, batch Numéro de lot de fabrication, lot Numero di lotto di produzione, partita Productielosnummer, batch</p> |  | <p>Medizinprodukt Medical device Dispositif médical Dispositivo medico Medisch hulpmiddel</p> |
|  | <p>CE-Kennzeichnung CE marking Marquage CE Marcatura CE CE-markering</p> |  | <p>Gebrauchsanweisung Instructions for use Mode d'emploi Istruzioni per l'uso Gebruiksaanwijzing</p> |
|  | <p>Eindeutige Gerätekenung Unique device identifier Identifiant unique de l'appareil Identificazione univoca del dispositivo Unieke apparaatidentificatie</p> |  | <p>einzelner Patient - mehrfach anwendbar single patient - multiple use patient individuel - utilisable plusieurs fois singolo paziente - utilizzabile più volte één patiënt - meerdere keren te gebruiken</p> |
|  | <p>Reinigung Cleaning Cleaning Pulizia Reiniging</p> |  | <p>Prothese Prosthesis Prothèse Protesi Prothese</p> |

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use


The nylon compression sheats serve as an intermediate layer between the stump and the socket and protect the skin from irritation when wearing a leg prosthesis.


2. Technical data

2.1 Size Selection


2.1.1 ST80 Nylon compression sheats for lower legs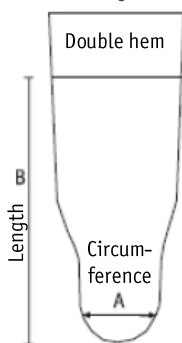

2.1.2 ST81 Nylon compression sheats for knee disarticulation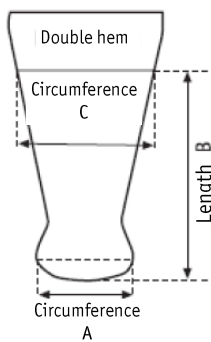

2.1.3 ST82 Nylon compression sheats for thighs

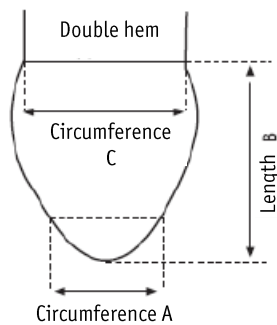2.2 ST80 Nylon compression sheats for lower legs

REF

| Order No. | Marking thread | Circumference A (cm) | Circumference C (cm) | Length B (cm) | Article No. |
|-----------|----------------|----------------------|----------------------|---------------|---------------------|
| ST80-1 | grey | 18-23 | --- | 20-32 | 4 236 200 01 00 000 |
| ST80-2 | light brown | 18-23 | --- | 32-48 | 4 236 200 02 00 000 |
| ST80-3 | yellow | 18-23 | --- | 45-64 | 4 236 200 03 00 000 |
| ST80-4 | light blue | 18-23 | --- | 60-87 | 4 236 200 04 00 000 |
| ST80-5 | reddish brown | 21-27 | --- | 20-32 | 4 236 200 05 00 000 |
| ST80-6 | blue | 21-27 | --- | 32-48 | 4 236 200 06 00 000 |
| ST80-7 | red | 21-27 | --- | 45-64 | 4 236 200 07 00 000 |
| ST80-8 | brown | 21-27 | --- | 60-87 | 4 236 200 08 00 000 |
| ST80-9 | black | 24-32 | --- | 20-32 | 4 236 200 09 00 000 |
| ST80-10 | dark green | 24-32 | --- | 32-48 | 4 236 200 10 00 000 |
| ST80-11 | dark grey | 24-32 | --- | 45-64 | 4 236 200 11 00 000 |
| ST80-12 | green | 24-32 | --- | 60-87 | 4 236 200 12 00 000 |

2.3 ST81 Nylon compression sheats for knee disarticulation

REF

| Order No. | Marking thread | Circumference A (cm) | Circumference C (cm) | Length B (cm) | Article No. |
|-----------|----------------|----------------------|----------------------|---------------|---------------------|
| ST81-1 | grey/yellow | 20-40 | 36-72 | 32-34 | 4 236 220 01 00 000 |
| ST81-2 | Green/grey | 20-40 | 36-72 | 34-40 | 4 236 220 02 00 000 |
| ST81-3 | grey/blue | 20-40 | 36-72 | 28-46 | 4 236 220 03 00 000 |

2.4 ST82 Nylon compression sheats for thighs

REF

| Order No. | Marking thread | Circumference A (cm) | Circumference C (cm) | Length B (cm) | Article No. |
|-----------|--------------------|----------------------|----------------------|---------------|---------------------|
| ST82-1 | dark brown/ yellow | 20-40 | 36-72 | 20-30 | 4 236 210 01 00 000 |
| ST82-2 | dark brown /green | 20-40 | 36-72 | 30-45 | 4 236 210 01 00 000 |
| ST82-3 | dark brown /blue | 20-40 | 36-72 | 45-70 | 4 236 210 01 00 000 |

3. Indications/ Contraindications

Indications:

- Treatment of a leg amputation

Contraindications:

- Extreme stump shape

4. Side effects

- There are no known side effects.

5. General safety instructions

- The medical device is intended for multiple use on one patient.
- The medical device may only be dispensed by qualified orthopaedic technicians.
- The technician must instruct the patient in the correct use of the product and inform them of any possible side effects/residual risks.
- The compression sleeves must not be used if there is any skin damage, injury or open wounds.
- If irritation, discomfort or poor circulation occurs, discontinue use of the product and consult your doctor immediately.
- When using a nylon compression sleeve, do not use oils or creams, deodorants, perfumes, bath additives, cleaning or alcohol sprays, or medically prescribed ointments, etc. for skin care of the amputation stump.
- When putting on the compression sleeves, make sure that no sharp-edged or rough objects come into contact with the compression sleeve.

6. Residual risks

Despite proper use of compression sheaths in a leg prosthesis, adverse circumstances may in some cases lead to instability (e.g. tripping, loss of balance, falling) as well as skin irritation and pressure points. To protect the skin, we recommend oil-free care products such as PS06 Allpremed® hydro INTENSIV. If you experience any adverse effects (e.g. skin irritation) during the prescribed use, please contact your doctor or orthopaedic specialist immediately.

7. Application information

- Ensure that you select the correct product according to the size and selection table.
- The nylon compression sheaths must not be too tight or too loose.
- Roll the compression sheath loosely outwards.
- Place the lower seam across the back of the stump and roll the compression sheath over the leg stump from below without pulling too hard. Thanks to the double elasticity of the compression sheath, it fits smoothly against the skin even with strong contours when the correct size is selected.
- If you have previously worn a wool stump sock, pull it over the compression sheath and roll both over the upper edge of the prosthesis.
- The compression sheath, pulled slightly upwards, should correspond to the length of the wool stump sock.

8. Maintenance and Cleaning

The nylon compression sheats must be checked regularly for damage and replaced if necessary.

- Do not wash the nylon compression sheat in the machine!
- Use a neutral mild detergent for washing.
- Do not rub or wring the nylon compression sheat vigorously.
- Hang the product on a line by the top to dry.
- It is recommended that the product be kept out of direct sunlight and away from heat sources.
- Please do not iron!
- Frequent washing does not reduce elasticity.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

Please observe the local disposal regulations.

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