Gebrauchsanweisung  
*Instruction Manual*  
Mode d'emploi  
*Istruzioni per l'uso*  
Gebruiksaanwijzing

**DAHO® Typ 3.1**

Version: 4, Stand: 30.09.2025

Uniprox GmbH & Co. KG  
H.-Heine-Str. 4  
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00  
Fax +49 (0) 36628-66-33 55  
E-Mail [info@uniprox.de](mailto:info@uniprox.de)

Symbole/ Icons/ Symboles/ Simboli/ Symbolen .....	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual .....	9
FR - Mode d'emploi .....	14
IT - Istruzioni per l'uso .....	19
NL - Gebruiksaanwijzing .....	24

## Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller  
Manufacturer  
Fabricant  
Produttore  
Fabrikant

Herstellungsdatum  
Date of manufacture  
Date de fabrication  
Data di produzione  
Productiedatum

Artikelnummer  
Article number  
Numéro d'article  
Numero articolo  
Artikelnummer

Achtung  
Attention  
A noter  
Attenzione  
Achtung

einzelner Patient - mehrfach anwendbar  
single patient - multiple use  
patient individuel - utilisable plusieurs fois  
singolo paziente - utilizzabile più volte  
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge  
Production lot number, batch  
Numéro de lot de fabrication, lot  
Numero di lotto di produzione, partita  
Productienummer, batch

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## EN - Instruction Manual

Please read the IFU carefully before fitting.  
Only correct usage will warrant the function.

### 1. Intended Use

Hand orthoses are used for immobilization, mobilization and/ or correction.

### 2. Technical data

Flexion orthosis with hook-and-loop fastene flexion cords

**REF**

Order No.	Size	Side	Article No.
DAH031-SL	S	left	4 695 010 01 01 000
DAH031-ML	M	left	4 695 010 01 02 000
DAH031-LL	L	left	4 695 010 01 03 000
DAH031-XLL	XL	left	4 695 010 01 04 000
DAH031-SR	S	right	4 695 010 02 01 000
DAH031-MR	M	right	4 695 010 02 02 000
DAH031-LR	L	right	4 695 010 02 03 000
DAH031-XLR	XL	right	4 695 010 02 04 000

### 2.1 Individual and spare parts

#### 2.1.1 Connection bar standard for *Intrinsic-plus-position*

**REF**

Order No.	Side	Article No.
E-DAH020	left	4 501 021 01 12 000
E-DAH021	right	4 501 021 02 12 000

#### 2.1.2 Forearm Cuff

**REF**

Order No.	Size	Side	Article No.
E-DAH023	Universal	universal	4 501 011 00 08 000

### 2.1.3 Midhand Cuff, two-piece

**REF**

Order No.	Size	Side	Article No.
E-DAH061	S	left	4 695 019 01 01 022
E-DAH062	M	left	4 695 019 01 01 033
E-DAH063	L	left	4 695 019 01 01 044
E-DAH081	XL	left	4 695 019 01 01 055
E-DAH064	S	right	4 695 019 02 01 022
E-DAH065	M	right	4 695 019 02 01 033
E-DAH066	L	right	4 695 019 02 01 044
E-DAH082	XL	right	4 695 019 02 01 055

### 2.1.4 MP-Blocker

**REF**

Order No.	Side	Article No.
E-DAH055	Left, for D2	4 695 019 01 03 000
E-DAH056	Right, for D2	4 695 019 02 03 000
E-DAH057	Left, for D3 – D5	4 695 019 01 04 000
E-DAH058	Right, for D3 – D5	4 695 019 02 04 000

### 2.1.5 Flexion band set incl. 10 pieces self-adhesive Velcro-pads

**REF**

Order No.	Side	Article No.
E-DAH059	universal	4 695 019 00 06 000

### 2.1.6 Connection bar long for "Kleinert"

**REF**

Order No.	Side	Article No.
E-DAH060	universal	4 695 019 00 02 000

### 2.1.7 Deviation hook, incl. screw

**REF**

Order No.	Side	Article No.
E-DAH067	universal	4 695 019 00 05 000

### 3. Indications/ Contraindications

Indications:

- Early movement treatment after flexor tendon surgery (in the intrinsic-plus position or according to Kleinert)
- Strengthening and stretching the extensor muscles of the fingers
- Redression of flexion deficits

Contraindications:

- Extreme deformities of the forearm or hand
- Extreme small or large hands
- Open wounds (without wound coverage)

### 4. Side effects

If worn for a long time, contact and the corrective effect of the orthosis can cause marks and irritation on the skin.

### 5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of the medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- The DAHO® Handorthosis is a prescription product that should be worn according the medical instructions.
- DAHO® Handorthosis should only be worn in accordance with these instructions and for the indications described.
- In the event of incorrect use product liability is excluded.
- In the event of over sensitivity of the skin, known contact allergies and sensitivity to pressure, a cotton glove should be worn under the DAHO® Handorthosis.
- No inexpert alterations may be made to the product. This may result in the performance of the product being limited, with the consequence that product liability is excluded.
- Patients should consult their doctor in the event of unusual changes in the hand function (e.g. an increase of the complaints).
- Combination with other products must be discussed with the patient's doctor beforehand.
- In the event of changes in performance or malfunctions of the orthosis, the patient should contact their orthopedic technician or doctor immediately.
- Avoid contact of the orthosis with wounds and skin deformations, without wound coverage!

### 6. Residual risks

When using the hand orthosis (outside of clothing) it can get caught/ tangled with other objects.

Despite proper use, skin irritation and pressure points can occur in individual cases.

## 7. Usage

The DAHO Hand Orthosis, Model 3.1 is supplied pre-assembled. The arm shell is connected to the two-part hand brace by means of a rigid connecting part. The two finger blockers (II and III-V) are flexed to 70° so that the hand can be positioned in the intrinsic plus position.

To position the hand according to Kleinert, the standard connecting part must be replaced with the long, flexed connecting part E-DAHO60. This must be ordered separately.

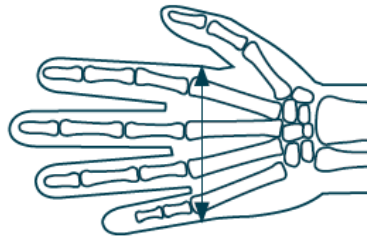
The thumb can also be flexed using the deflection hook. The dynamic thumb extension (DAHO-EX left or right) can be used without the spring balancer to block the movement of the thumb.

### 7.1 Size selection

To select the right orthotic size, the metacarpal width is measured at the level of the MCP joints.

Size	Midhand width
S	5,5 – 7,0 cm
M	7,1 – 8,5 cm
L	8,6 – 11,0 cm
XL	9,0 – 11 cm for powerful hands

### 7.2 Adjustment

The orthosis is worn on the skin. According to the specific condition, the orthosis must be individually adapted either with thermoplastic or a mechanical conversion.

- Remove the blockers from the metacarpal brace, open all hook-and-loop fasteners and loosen the screws on the connecting part.
- Put on the arm brace and the metacarpal brace, adjust the length and tighten the screws on the connecting part. The position of the wrist can be modified by adjusting the connecting part.
- Close the hook-and-loop fastener.
- Adjust required blockers and screw them onto the metacarpal brace.
- Clean the fingernail with a disinfectant and stick the supplied adhesive tape on the fingernail.
- Fasten the elastic straps and identify the length to pad stud. Tie a loop according to the required tensile force and hook the loop into the pad stud.

## 8. Maintenance and Cleaning

The orthosis can be washed with mild soap and hand warm water.  
Commercially available disinfectants should be used for surface disinfection.

## 9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed by Uniprox and the competence European Authority.

## 10. Warranty and Service life

We recommend to change/ renew the hand orthosis after 6 month.

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

## 11. Storage and Disposal

The orthosis has no special storage regulations. Please note the country-specific disposal.

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