Gebrauchsanweisung
Instruction Manual
Mode d'emploi
Istruzioni per l'uso
Gebruiksaanwijzing

DAHO® Typ 3

Version: 6, Stand: 30.09.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	8
FR - Mode d'emploi	12
IT - Istruzioni per l'uso	16
NL - Gebruiksaanwijzing	20

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

	<p>Hersteller Manufacturer Fabricant Produttore Fabrikant</p>		<p>Herstellungsdatum Date of manufacture Date de fabrication Data di produzione Productiedatum</p>
	<p>Artikelnummer Article number Numéro d'article Numero articolo Artikelnummer</p>		<p>Achtung Attention A noter Attenzione Achtung</p>
	<p>einzelner Patient - mehrfach anwendbar single patient - multiple use patient individuel - utilisable plusieurs fois singolo paziente - utilizzabile più volte één patiënt - meerdere keren te gebruiken</p>		<p>Fertigungslosnummer, Charge Production lot number, batch Numéro de lot de fabrication, lot Numero di lotto di produzione, partita Productienummer, batch</p>
	<p>CE-Kennzeichnung CE marking Marquage CE Marcatura CE CE-markering</p>		<p>Medizinprodukt Medical device Dispositif médical Dispositivo medico Medisch hulpmiddel</p>
	<p>Eindeutige Gerätekenung Unique device identifier Identifiant unique de l'appareil Identificazione univoca del dispositivo Unieke apparaatidentificatie</p>		<p>Gebrauchsanweisung Instructions for use Mode d'emploi Istruzioni per l'uso Gebruiksaanwijzing</p>
	<p>Reinigung Cleaning Cleaning Pulizia Reiniging</p>		<p>Immobilisierung Immobilization Immobilisation Immobilizzazione Immobilisatie</p>
	<p>Korrektur Correction Correction Correzione Correctie</p>		<p>Mobilisierung Mobilization Mobilisation Mobilizzazione Mobilisatie</p>

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.

1. Intended Use

Hand orthoses are used for immobilization, mobilization and/ or correction.

2. Technical data

Flexion Orthosis with glove

Order No.	Size	Side	Article No.
DAH03-SL	S	left	4 501 070 01 01 000
DAH03-ML	M	left	4 501 070 01 02 000
DAH03-LL	L	left	4 501 070 01 03 000
DAH03-SR	S	right	4 501 070 02 01 000
DAH03-MR	M	right	4 501 070 02 02 000
DAH03-LR	L	right	4 501 070 02 03 000

2.1 Individual and spare parts

2.1.1 Connection bar standard

Order No.	Side	Article No.
E-DAH020	left	4 501 021 01 12 000
E-DAH021	right	4 501 021 02 12 000

2.1.2 Forearm Cuff

Order No.	Size	Side	Article No.
E-DAH023	Universal	universal	4 501 011 00 08 000

2.1.3 Midhand Cuff, one piece

REF

Order No.	Size	Side	Article No.
E-DAH037	S	left	4 501 021 01 08 000
E-DAH038	M	left	4 501 021 01 09 000
E-DAH039	L	left	4 501 021 01 10 000
E-DAH040	S	right	4 501 021 02 08 000
E-DAH041	M	right	4 501 021 02 09 000
E-DAH042	L	right	4 501 021 02 10 000

2.1.4 Glove with rubber bands

REF

Order No.	Size	Side	Article No.
E-DAH049	S/M	left	4 501 071 01 04 000
E-DAH050	L	left	4 501 071 01 05 000
E-DAH051	S/M	right	4 501 071 02 04 000
E-DAH052	L	right	4 501 071 02 05 000

3. Indications/ Contraindications

Indications:

- Redressment of flexion deficits and soft-tissue contractures
- Strengthening extensor muscles of fingers by dynamic

Contraindications:

- Extreme deformities of the forearm or hand
- Extreme small or large hands
- Open wounds (without wound coverage)

4. Side effects

If worn for a long time, contact and the corrective effect of the orthosis can cause marks and irritation on the skin.

5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of the medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- The DAHO® Hand orthosis is a prescription product that should be worn according to the medical instructions.
- DAHO® Hand orthosis should only be worn in accordance with these instructions and for the indications described.
- In the event of incorrect use product liability is excluded.
- In the event of over sensitivity to the skin, known contact allergies and sensitivity to pressure, a cotton glove should be worn under the DAHO® Hand orthosis.

- No inexpert alterations may be made to the product. This may result in the performance of the product being limited, with the consequence that product liability is excluded.
- Patients should consult their doctor in the event of unusual changes in the hand function (e.g. an increase of the number of complaints).
- Combination with other products must be discussed with the patient 's doctor beforehand.
- In the event of changes in performance or malfunctions of the orthosis, the patient should contact their orthopedic technician or doctor immediately.
- Avoid contact of the orthosis with wounds and skin deformations, without wound coverage!

6. Residual risks


When using the hand orthosis (outside of clothing) it can get caught/ tangled with other objects. Despite proper use, skin irritation and pressure points can occur in individual cases.

7. Usage

7.1 Size selection

To select the right orthotic size, the metacarpal width is measured at the level of the MCP joints.

Size	Metacarpal width
S	5,5 – 7,0 cm
M	7,1 – 8,5 cm
L	8,6 – 11,0 cm

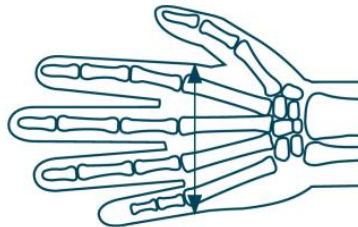7.2 Adjustment

The orthosis is worn on a thin glove.

According to the specific condition, the orthosis must be individually adapted either with thermoplastic or a mechanical conversion.

- The midhand cuff and the forearm cuff are positioned accordingly and closed with a hook-and-loop fasteners. The midhand cuff should not extend the proximal palm fold.
- The glove with the rubber bands is positioned on the patient.
- The rubber bands of the glove are threaded between the metacarpus clip and the diverter pulley.
- The amount of pull is determined by the length and tension of the rubber bands and adjusted according to the situation and the instructions of the doctor or therapist.

8. Maintenance and Cleaning

The orthosis can be washed with mild soap and hand warm water.
Commercially available disinfectants should be used for surface disinfection.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

10. Warranty and Service life

We recommend to change/ renew the hand orthosis after 6 month.

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

The orthosis has no special storage regulations.

Please note the country-specific disposal.

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